

Caution: Do not steam clean or autoclave the cushion. The cushion cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the cushion and/or cushion cover will age more rapidly.

Checking for wear and deflated cells

Every three months, check the cushion for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the cushion and contact Morton & Perry.

Cleaning the inner cushion

- Do not immerse the inner cushion in water or any other liquid.
- Do not use aggressive or abrasive cleaners, or sharp objects to clean the inner cushion.
- Treat light stains by wiping the inner cushion with a damp cloth, using a mild detergent.
- Difficult spots or stains can be cleaned by first saturating the spot and then using a soft brush to loosen the dirt, or by machine washing the emptied inner cover at 60°.
- If it is necessary to remove the cells for cleaning, note the number of cells in each compartment.

Cleaning the outer cover

The outer cover can be machine-washed inside out. See the instructions on the label attached to the inside of the cover.

Disinfecting the cushion

In case of soiling with body fluids, and when transferring the cushion to another user, the cushion must be disinfected to avoid cross contamination.

- Clean the inner cushion and the outer cover as described above. When washing the outer cover, preferably use an anti-bacterial agent.
- Spray disinfectant on the outer cover and the inner cushion. Bend the compartments outwards to make sure that the spray mist gets into the gaps.

Warranty

Vicair offers a limited warranty against defects in workmanship and materials for a period of two (2) years after the original date of purchase, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty. Should a defect in materials or workmanship occur within two (2) years from the original date of purchase, Vicair will, at its own choice, repair or replace the product free of charge. The outer cover is excluded from this warranty. Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.